



Silente™ adjustable venturi valve

Low operating noise



Oxygen and Aerosol Therapy • Fixed Oxygen Concentration (High Flow)

Silente adjustable venturi valve

The Silente has been designed to operate at a very low noise level. The large diameter body provides space for clear, widely spaced calibration and incorporates a "pull-twist" action to prevent accidental adjustment.

The twin-jet design offers the user seven oxygen concentrations. The increased body diameter provides space for clear, widely spaced calibration and incorporates a 'pull-twist' action preventing accidental adjustment. The one-piece design eliminates the need to use more than one valve for all concentrations and there are no separate pieces to get lost or mislaid.

The concentration delivered is unaffected by the patient's breathing pattern provided that the optimum flow indicated on the valve is set on the oxygen flow meter.

Make an enquiry

Incurved nose seal

Conforms to different nose shapes designed to prevent oxygen entering patient's eyes

Large, well spaced calibration settings

Easy to read, minimises errors

One piece design

No parts to change or lose

Extremely quiet

Much improved patient environment

Ease of use

The Intersurgical adjustable venturi valve is simple to use and has the following features and benefits:

- The appropriate flow range is selected to suit the desired concentration
- The outer ring with its pull-twist action selects the prescribed oxygen concentration
- A flow in L/min is printed above each concentration indicating the optimum flow meter setting.

Precision

The Silente delivers precise oxygen concentrations. This accuracy is independent of the oxygen flow meter setting and is achieved using just one valve.



Soft face seals

Increased patient comfort

Elastic positioned under the ears

Eliminates trauma to top of ears

Pull-Twist controls

Prevent inadvertent adjustment

Code	Description	Box Qty.
1137000	Silente, Eco, adult adjustable venturi mask kit	80
1138015	Silente, Eco, paediatric adjustable venturi mask kit	40
1198015	Intersurgical EcoLite™, paediatric aerosol mask	42
1188000	Eco, adult, aerosol mask	70
1139000	Silente, adjustable venturi valve only	50

Lower environmental impact product



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK
 T: +44 (0)118 965 6300 F: +44 (0)118 965 6356 info@intersurgical.com www.intersurgical.com



The manufacturer Intersurgical Ltd is certified to ISO 9001:2015, ISO 13485:2016 and ISO 14001:2015

Please think before you print
 Save energy and paper.
 If you must print this information sheet please print it double sided.

IS10.21 • Issue 6 09.19