



# STARMed Ventukit Up

Openable CPAP hood with built-in Venturi flow driver



Critical Care ▪ Patient Interfaces ▪ STARMed



Quality, innovation and choice

Interact with us



[www.intersurgical.co.uk](http://www.intersurgical.co.uk)

# STARMed Ventukit Up openable CPAP hood with Venturi flow driver


## StarMed Ventukit Up

Openable Ventukit Up is a complete and easy to use system that is simply activated by connecting to an appropriate oxygen supply. It can be used to deliver CPAP in a hospital environment, transport or pre-hospital use.

The openable Ventukit Up allows for one operator to easily remove the upper part of the hood and have full access to the patient, whilst any probes or catheters can remain in position on the lower part of the hood.

## Technical Features

- The integrated Venturi flow driver can generate high patient flows while consuming a reduced amount of compressed oxygen
- O<sub>2</sub>% can be adjusted from 40% to 100% at each PEEP level
- The adjustable PEEP valve can provide patient pressures in a range of settings up to 20 cmH<sub>2</sub>O
- The user is guided by a performance table integral to the product, for independent PEEP and O<sub>2</sub>% settings

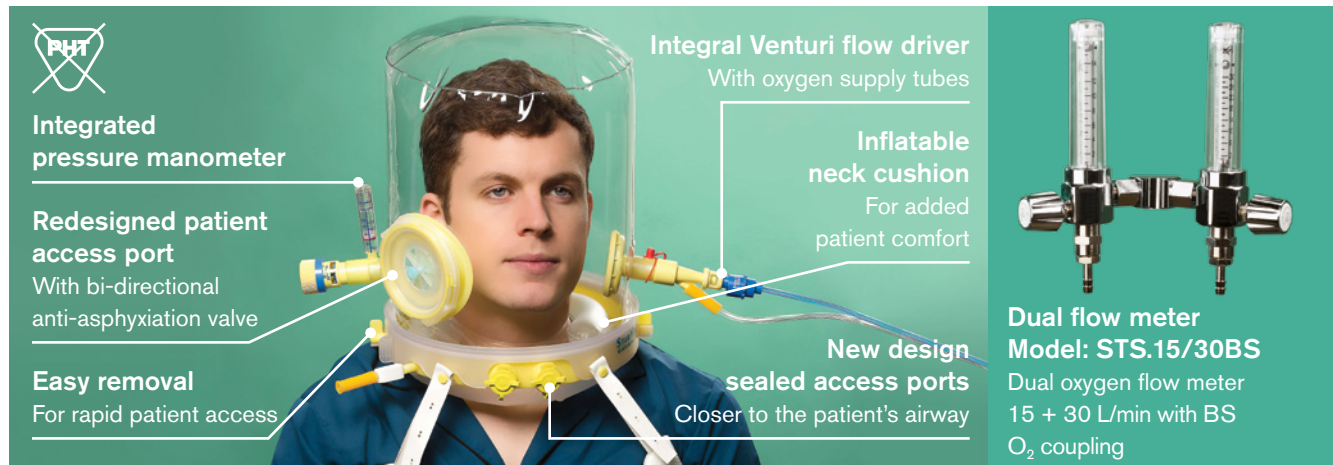

## Ventukit Up openable CPAP hood with Venturi flow driver, with bi-directional anti-asphyxiation valve and patient access port, inflatable neck cushion and inflation bulb, adjustable PEEP valve, ear plugs and measuring tape

Requirement		Setting	Flow delivered
O <sub>2</sub> % approx	PEEP cmH <sub>2</sub> O	Flow O <sub>2</sub> L/min A + B	Flow approx L/min
40%	5	9 + 0	46
	7.5	9 + 0	47
	10	10 + 0	48
	12.5	10 + 0	46
	15	11 + 0	47
50%	20	12 + 0	48
	5	8 + 9	47
	7.5	8 + 9	47
	10	9 + 9	45
	12.5	9 + 8	48
	15	10 + 8	50
	20	12 + 9	54

Requirement		Setting	Flow delivered
O <sub>2</sub> % approx	PEEP cmH <sub>2</sub> O	Flow O <sub>2</sub> L/min A + B	Flow approx L/min
60%	5	7 + 16	52
	7.5	8 + 16	48
	10	8 + 14	48
	12.5	9 + 13	47
	15	9 + 13	47
	20	10 + 14	47
100%	Feed with a minimum O <sub>2</sub> flow of 40 L/min		

Performance table subject to changes. Always refer to product labelling.

Code	Size	Neck circumference	Box Qty.
CVA106XXL/2W	Extra Extra Large	≥50cm	5
CVA106XL/2W	Extra Large	45-52cm	5
CVA106L/2W	Large	40-47cm	5
CVA106M/2W	Medium	34-41cm	5
CVA106S/2W	Small	27-34cm	5
CVA106XS/2W	Extra Small	17-27cm	5



[www.intersurgical.co.uk/info/starmed](http://www.intersurgical.co.uk/info/starmed)

Make an enquiry

Dual flow meter manufactured by FLOW METER S.p.A.



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK  
T: 0118 965 6300 F: 0118 965 6356 info@intersurgical.co.uk www.intersurgical.co.uk



Manufacturer:  
Intersurgical S.p.A.  
Mirandola (MO) Italy

**Please think before you print**  
Save energy and paper. If you must print this information sheet please print it double sided.

IS1.46\_UK • Issue 3 06.20